


Baxter

Peri-Strips Dry

STAPLE LINE REINFORCEMENT
WITH VERITAS COLLAGEN MATRIX
WITH SECURE GRIP TECHNOLOGY

Preparation Instructions

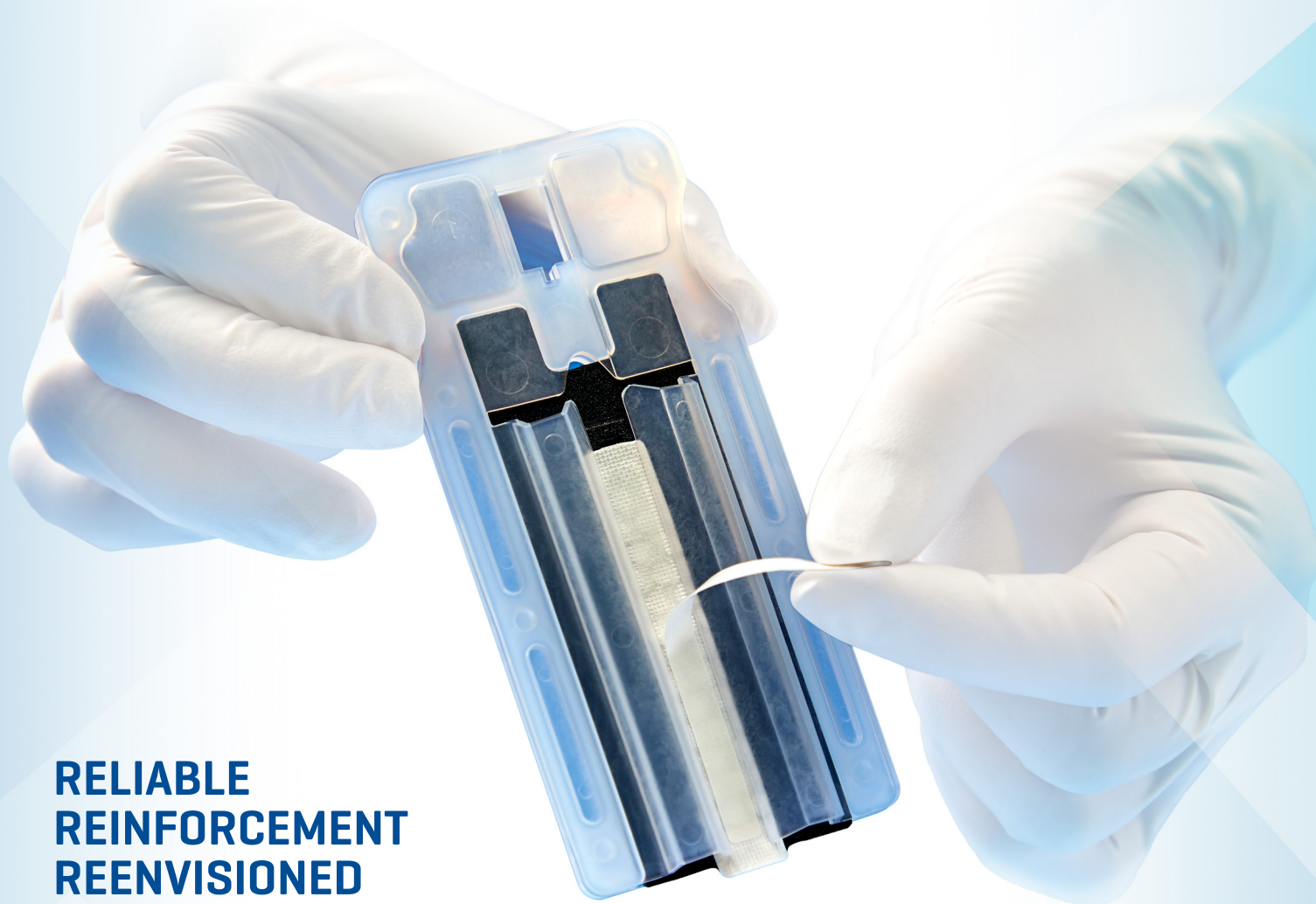

**RELIABLE
REINFORCEMENT
REENVISIONED**



Peri-Strips Dry

STAPLE LINE REINFORCEMENT WITH VERITAS COLLAGEN MATRIX WITH SECURE GRIP TECHNOLOGY

Each PERI-STRIPS DRY with VERITAS with Secure Grip Technology (PSDV Secure Grip) loading unit is packaged sterile in a separate pouch. PSDV Secure Grip is comprised of a loading unit, two bovine pericardium buttress strips, and a plastic sheath.

PSDV Secure Grip is secured to the linear surgical stapler via an adhesive strip. The user must pull the liner from both the Anvil and Cartridge sides of the product to expose the adhesive before placing the product on the stapler.

Before opening any PSDV Secure Grip product, check the label to ensure that the product is designed for the stapler of choice.

1. OPEN



- Inspect outer pouch. Do not use product if pouch integrity is compromised.
- In a sterile field, open inner pouch using aseptic technique.



- Remove and discard ship clip.

2. PEEL



- Peel liner from both from both Anvil and Cartridge sides.

Note: The plastic sheath is clearly labeled ANV depicting the Anvil side and CART depicting the Cartridge side. These labels reflect the which sides of the product correspond to the stapler jaws.

3. SECURE




- Position open stapler jaws with corresponding sides of the loading unit and advance into the stapler jaws until fully seated.
- Close stapler.



- Remove and discard plastic sheath.

4. PASS

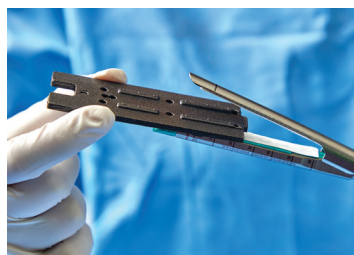

- Open the stapler jaws and remove foam insert.
- Visually inspect each buttress strip to ensure it adheres to the stapler and covers the staple line.



- Close stapler and pass to surgeon.

Indications and Important Risk Information for PERI-STRIPS DRY with VERITAS Collagen Matrix with Secure Grip Technology

PERI-STRIPS DRY with VERITAS Collagen Matrix with Secure Grip Technology Indications for Use

PERI-STRIPS DRY with VERITAS Collagen Matrix with Secure Grip Technology (PSDV Secure Grip) is intended for use as a prosthesis for the surgical repair of soft tissue deficiencies using surgical staplers when staple line reinforcement is needed.

PSDV Secure Grip can be used for reinforcement of staple lines during gastric, bariatric, small bowel, colon and colorectal procedures.

Contraindications

The use of PSDV Secure Grip is contraindicated in patients with known sensitivity to bovine or acrylic material.

Warnings and cautions

Do not re-sterilize.

Do not use PSDV-SG if the product has been exposed to solutions above room temperature, to chemicals, antibiotics or other substances other than specifically addressed in the instructions, or if the heat indicator has been activated.

The use of PSDV-SG has not been studied in lung or bronchus resection.