





Vascu-Guard

VASCULAR REPAIR PATCH

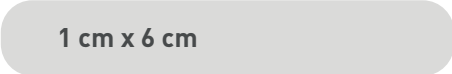

SCAN FOR DETAILS

PRODUCT SIZE [PATCHES REPRESENT ACTUAL SIZE]	ORDER CODE	QTY.
 1 cm x 6 cm	VG0106	1 EA
 0.8 cm x 8 cm (tapered end)	VG0108	1 EA
 1 cm x 10 cm	VG0110	1 EA
 2 cm x 9 cm	VG0209	1 EA

VASCU-GUARD Vascular Repair Patch is used in peripheral vascular reconstruction including the carotid, renal, iliac, femoral, profunda and tibial vessels and arteriovenous access revisions.

Please see Important Risk Information on back.

Baxter

VASCU-GUARD INDICATIONS FOR USE

VASCU-GUARD Vascular Repair Patch is used in peripheral vascular reconstruction including the carotid, renal, iliac, femoral, profunda and tibial vessels and arteriovenous access revisions.

CONTRAINDICATIONS

VASCU-GUARD is not designed, sold or intended for use except as indicated.

Do not use VASCU-GUARD in patients with known sensitivity to bovine material.

WARNINGS

Failure to rinse the product may result in a sterile inflammatory reaction in the adjoining host tissue.

The product must remain moist at all times.

Do not place outer pouch in sterile field as outside of pouch is not sterile.

Do not use if any folds, creases, or wrinkles are observed in VASCU-GUARD patch.

Do not use if product has been exposed (1) to solutions above room temperature or (2) to chemicals, antibiotics, or substances other than specifically addressed in these instructions as the characteristics of VASCU-GUARD may change or be compromised.

Antimycotics (anti-fungals) must not come in contact with VASCU-GUARD as they are believed to alter the cross-linked characteristics of tissue fixed in aldehyde preparations.

Do not resterilize.

Product is single use only.

Failure to observe warnings may result in surgical infection.

Do not substitute one Synovis product for another.

Clinical experience with glutaraldehyde fixed porcine xenograft heart valves indicates that fixed tissue may be subject to late attack by the body and subsequent tissue deterioration. In like manner, the glutaraldehyde fixed bovine pericardium may be subject to late deterioration. The benefits of the use of this tissue in cardiovascular repair or repair of soft tissue deficiencies must be weighed against the possible risk of aneurysm, hemorrhage, or patch weakening resulting from tissue deterioration.

The patch is not indicated for the construction or replacement of total valves or conduits.

Rinse surgical gloves to remove glove powder prior to touching VASCU-GUARD.

Rx Only. For safe and proper use of this device refer to the complete Instructions for Use.